

# Oasis TEARS™

## Preservative-free, Viscoadaptive, Lubricating Eye Drops

Oasis TEARS™ viscoadaptive lubricating eye drops coat, lubricate and moisten delicate ocular tissue. With each blink of the eye, this unique drop coats and re-coats the surface of the eye for lasting relief of symptoms.

- Preservative-free
- Instant relief
- Fewer applications required
- 30 sterile, disposable containers per box

### Why preservative-free?

Most artificial tears contain preservatives which have recently been discovered to be the culprit in damaging the top layer of the cornea leading to disruption of the tear film when drops are used frequently. This sequence leaves the surface of the eye unable to keep tear film in place and can lead to ocular surface disease. Using more than 4 to 6 drops per day increases the likelihood of preservative-induced adverse effects.



**REF# OT6100**

Recommended for mild to moderate symptoms



**REF# OT6200**

Recommended for moderate to severe symptoms

For more information on Oasis TEARS™  
or to place an order:

**800.631.7180**  
**www.oasistears.com**

## DRY EYE SOLUTIONS

Whether your patients suffer from chronic Dry Eye or post-surgical temporary Dry Eye, OASIS artificial tears and punctal occlusion products can help you treat them.

**OASIS®** 

514 S. Vermont Avenue  
Glendora, CA 91741 USA

800.528.9786  
909.305.5400

800.631.7210  
909.305.9987 FAX

[www.oasismedical.com](http://www.oasismedical.com)

## DRY EYE SOLUTIONS


FROM OASIS® 



Soft Plug®  
Silicone



OasisTEARS™

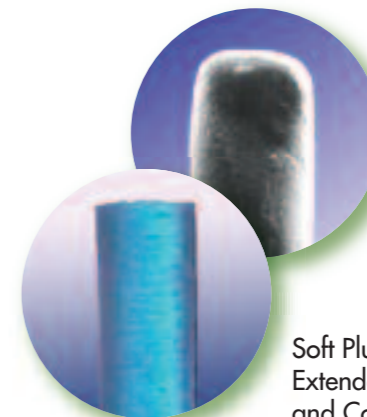

Soft Plug®  
Extended Duration  
and Collagen



Form Fit® One Size  
One Solution Hydrogel

# FORM FIT®

## Hydrogel Intra Canalicular Plug



Form Fit® is truly a one size, one solution plug. Once inserted into the punctum, Form Fit® hydrates when it contacts tear film into a soft, pliable, gelatinous material. As it hydrates, it increases in size until it completely fills the vertical canalicular cavity to form a perfect fit. There is no need for a cap or anchor at the end of the plug.


Form Fit® comes preloaded in an inserter. The polyimide sheath is inserted into the punctum and the inserter handle compressed to expel the hydrogel plug.

To remove the Form Fit® plug, simply flush saline solution through the punctal opening. The gelatinous material will be expressed through the nasal passage.


### PRODUCT FEATURES

- In its dry state, the material is a rigid rod measuring 0.3mm in diameter by 3.0mm long
- The Form Fit® plug requires no special storage
- The need for punctal measurement has been eliminated
- Hydrogel plug is biocompatible and has no foreign body sensation
- Available pre-loaded in boxes of 2 or 10 plugs

Patent # 5,116,37

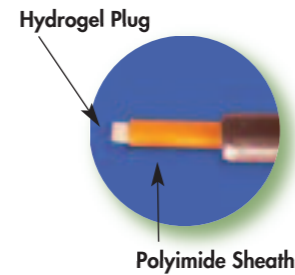

### INSERTION

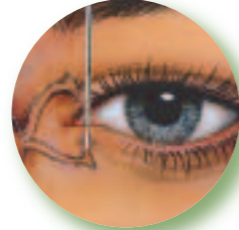

### IMMEDIATELY AFTER INSERTION



### 10 MINUTES AFTER INSERTION



## SOFT PLUG® Silicone Punctum Plugs

The distinctive low profile cap combined with a softer, more flexible silicone maximizes patient comfort. The pointed nose design simplifies insertion while the large anchor with wide shelf firmly secures the plug.

The inserter provides greater control and eliminates accidental release of the plug. SOFT PLUG® Silicone Plugs are removable.

Available preloaded on inserter in 2-pack box or 6-pack dispenser box.

Also available in non-loaded Economy Pack: Includes 2 inserters and 10 trays of 2 plugs each. Trays equipped with an alignment slot for quick loading.

### Available Sizes



- 0.4mm Micro
- 0.5mm Mini
- 0.6mm Petite
- 0.7mm Small
- 0.8mm Medium

## Extended Duration Plugs

Extended Duration absorbable plugs are designed to provide patient comfort for up to 3 months. This long-lasting plug is effective and economical for transient dry eye symptoms following surgery. Extended Duration plugs are 2mm in length.

### Diameter Sizes

- 0.2mm, 0.3mm, 0.4mm and 0.5mm

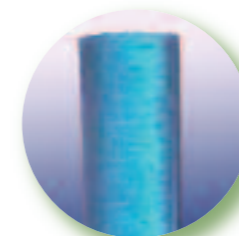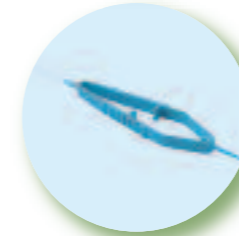

## SOFT PLUG® Collagen Intra Canalicular Plugs

The collagen intra canalicular plugs have a smooth proprietary finish that simplifies insertion of the plug. Collagen plugs are ideal for short-term, diagnostic and post-surgical occlusion. Plugs are 2mm in length and absorbable within 2-5 days.

### Diameter Sizes

- 0.2mm, 0.3mm and 0.4mm

## Instruments

### Sizing Gauges

Facilitate accurate punctal measurement prior to plug insertion. Gauges are bar coded for quick size identification.

- Sizes: 0.5mm and 0.6mm tips
- 0.7mm and 0.8mm tips

### Castroviejo Lacrimal Dilator

### Jeweler's Forceps

Types: Straight #3 and Curved #7

### Grooved Forceps

Aids insertion of Extended Duration Plugs

## Tear Tests

**ZONE-QUICK** Phenol Red Thread Test

**TEAR-FLO** Test Strips

## Patient Education Support Materials

- Dry Eye Patient Brochure
- Surgical Dry Eye Patient Brochure
- Dry Eye Videos: Patient Education and Punctal Occlusion Training
- Dry Eye Wall Chart

